



# STOMA PROTECTIVE POWDER (NON-STERILE)

keeps skin dry, and reduces irritation from exudate.



# STOMA PROTECTIVE POWDER (NON-STERILE)

## Stoma Protective Powder:

It is intended for use on the peristomal skin to keep it dry, which in turn reduces the irritation of the skin by exudate.

Patients with the ostomy such as colostomy, jejunostomy, ileostomy.

## Feature:

- Rapidly absorbs moisture on the skin's surface to keep it dry.
- Can be used between the peristomal skin and skin barrier to form a thin protective film on the skin's surface.
- Shows excellent properties of biocompatibility, being odorless and highly hygroscopic.

## SPECIFICATIONS


REF. No.	Bottle	Description
SY02050	50 g	- Non-sterile



SY02050

## HOW TO USE

Patients with the ostomy such as colostomy, jejunostomy, ileostomy.

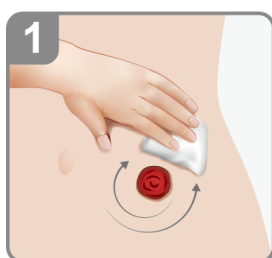

### CLEAN

Clean and dry the peristomal skin.



### SHAKE

Shake the bottle before opening.



### APPLY

Aim the nozzle at the intended site and apply powder by shaking the bottle.



### WIPE

Wipe off extra powder gently to form a thin protective film on the skin.



**fortune**  
medical

6 F., No. 29, Sec. 2, Jhongjheng E. Rd.,  
Danshuei Dist, New Taipei City. 251 Taiwan  
Tel.:(886) 2-2624-2233 Fax.:(886) 2-2624-2266  
www.fortunemed.com Mail:info@fortunemed.com